




PRR & SMALL CLASS I RESTORATION

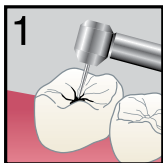


1
Prepare tooth as needed,
rinse, leave damp.

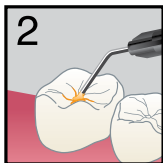


2
Apply **BEAUTIFIL kids SA**
in a thin layer ($\leq 0.5\text{mm}$)
with needle tip or hand
instrument.
Leave for 20 sec.

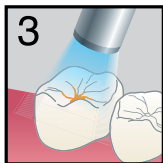


3
Light cure for 5 sec (LED).

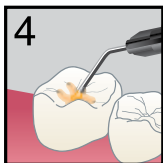

4
Apply additional
increments ($\leq 2\text{mm}$) as
needed.

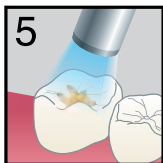

5
Light cure each increment
for 10 sec (LED).

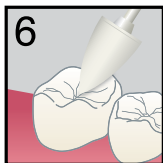

6
Finish and polish with
OneGloss PS or another
polisher.

Visit shofu.com Call 800.827.4638

Shofu Dental Corporation San Marcos, CA